



COVID-19 SHIELD

SALIVA SAMPLE

- Remove the tube lid

- Place the funnel on the top of the tube

- Collect saliva in your mouth

- Place funnel at your mouth

- Dribble saliva through the funnel into the tube

- Leave funnel on tube, repeat until **2mL** of saliva (**not bubbles**) is collected

- Remove funnel from tube

- Throw away funnel in designated biohazard waste container

- Close the lid tightly on the tube

- Take your sealed tube to the collection station to be verified

- Under the **CARES Act** all test results are **shared** with the Illinois Department of Public Health



richland.edu/shield



NO PHOTOGRAPHY



NO VIDEO